**TREATMENT AND PREVENTION FOR DUST MITE ALLERGY**

✓ Relieves allergic symptoms by attracting the dust mites & allowing you to eliminate them

**INSTRUCTIONS FOR USE****PREPARE THE MATTRESS AND PILLOWS**

- Remove comforters and mattress protectors from the mattress
- Cover the entire mattress and pillows with a textile\*  
\*Different types of textile can be used (100% cotton) such as bed linen and towels.

**TO ATTRACT THE DUST MITES INTO THE TEXTILE**

- Spray the ExAller® solution evenly over the entire surface of the textile being used (single bed mattress: 24 sprays; double bed mattress: 48 sprays; per pillow: 3 sprays)
- Remove the textile after minimum one hour and maximum two hours

**TO REMOVE THE DUST MITES OUT OF THE TEXTILE**

- Wash the textile in the washing machine (60°C) – do not use fabric conditioner

**TO REMOVE THE ALLERGENS FROM THE MATTRESS AND PILLOWS**

- It's important to vacuum the mattress and pillows after using ExAller® for the first three applications

**DOSAGE**

**TO ELIMINATE THE DUST MITES:** use once a week for the first three weeks

**TO PREVENT FURTHER OCCURENCE:** use once a month

**PRODUCT FORMATS**

- 75 ml: 4 applications for a single bed
- 150 ml: 4 applications for a double bed or 8 applications for a single bed
- 300 ml: 8 applications for a double bed or 16 applications for a single bed

**COMPOSITION**

Solution based on natural plant extract

**STORAGE CONDITIONS**


ExAller® solution must be stored at a temperature between 5°C and 25°C

**WARNING**

- Shake the ExAller® solution well before each use.
- When first using ExAller® solution remove the aluminium seal and attach the spray.
- Do not use if the packaging is damaged.
- Do not leave within the reach of children.
- Avoid any contact with the eyes.
- Do not use after the expiry date indicated on the bottle.
- This product has not been tested in children under 3 years of age as well as pregnant or lactating women.
- Any serious incident related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the patient is established.
- If symptoms persist, consult your doctor.

For more information: [www.exaller.com](http://www.exaller.com)

Acar'Up Consumer Health SPRL  
Clos Chapelle-aux-Champs, 30  
1200 BRUSSELS  
BELGIUM

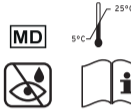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