TREATMENT AND PREVENTION FOR DUST MITE ALLERGY

✓ Relieves allergic symptoms by attracting the dust mites & allowing you to eliminate them

INSTRUCTIONS FOR USE

PREPARE THE MATTRESS AND PILLOWS

- Remove comforters and mattress protectors from the mattress
- Cover the entire mattress and pillows with a textile*
*Different types of textile can be used (100% cotton) such as bed linen and towels.

TO ATTRACT THE DUST MITES INTO THE TEXTILE

- Spray the ExAller® solution evenly over the entire surface of the textile being used (single bed mattress: 24 sprays; double bed mattress: 48 sprays; per pillow: 3 sprays)
- Remove the textile after minimum one hour and maximum two hours

TO REMOVE THE DUST MITES OUT OF THE TEXTILE

- Wash the textile in the washing machine (60°C) – do not use fabric conditioner

TO REMOVE THE ALLERGENS FROM THE MATTRESS AND PILLOWS

- It's important to vacuum the mattress and pillows after using ExAller® for the first three applications

DOSAGE

TO ELIMINATE THE DUST MITES: use once a week for the first three weeks

TO PREVENT FURTHER OCCURENCE: use once a month

PRODUCT FORMATS

75 ml: 4 applications for a single bed
150 ml: 4 applications for a double bed
or 8 applications for a single bed
300 ml: 8 applications for a double bed
or 16 applications for a single bed

COMPOSITION

Solution based on natural plant extract

STORAGE CONDITIONS

ExAller® solution must be stored at a temperature between 5°C and 25°C

WARNING

- Shake the ExAller® solution well before each use
- When first using ExAller® solution remove the aluminium seal and attach the spray
- Do not use if the packaging is damaged
- Do not leave within the reach of children
- Avoid any contact with the eyes
- Do not use after the expiry date indicated on the bottle
- This product has not been tested in children under 3 years of age as well as pregnant or lactating women
- Any serious incident related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the patient is established
- If symptoms persist, consult your doctor

For more information: www.exaller.com

Acar'Up Consumer Health SPRL
Clos Chapelle-aux-Champs, 30
1200 BRUSSELS
BELGIUM

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